


HOOFSAVERS™



Before Use

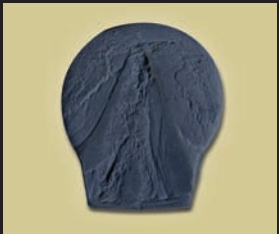

After Use

- Use daily to provide relief to sore-footed, stone-bruised and laminitic horses
- Use as a preventive measure against foot ailments
- Provide protection to shoeless horses
- Perfect for use at shows, overnight or even during shipping
- Safe for use with medication


CUSHIONED COMFORT. FULL-SOLE SUPPORT.

USING EVA FOAM TECHNOLOGY SIMILAR TO THAT USED IN THE RUNNING SHOE INDUSTRY, EQUIFIT'S CUSTOM-FIT HOOFSAVERS PROVIDE CUSHIONED COMFORT AND FULL-SOLE SUPPORT FOR YOUR HORSE'S HOOVES

DIRECTIONS FOR USE:



Stand horse on pads for 60 seconds or more...

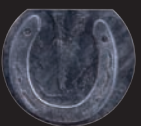

to make a solid impression.



Using scissors, cut along edge of impression line. For shod hooves, cut along inside edge of shoe impression. Shoeless hooves should have padding to the outer edge of the hoof.



Remove any dirt and debris in the hoof and apply medication to the sole if needed. Place HoofSaver in the sole. Adhere custom HoofSaver to the hoof using Vetwrap, duct tape or similar.



HoofSaver may be left in position for up to 24-hours* or as long as horse is in stall. To remove, simply release tape and pull HoofSaver from hoof.



Brush off and clean HoofSaver. Re-apply as needed. HoofSaver may lose some of its original impression when not in use; once re-applied it will conform back to the sole for a custom fit. Good for several uses.

**Remove pads when riding or exercising the horse. Never use more than one layer of pads per hoof. Not intended for use on multiple horses.*

FOR MORE INFORMATION OR TO RE-ORDER, VISIT YOUR
LOCAL TACK STORE OR WWW.EQUIFIT.NET

EquiFit, inc.

BETTER FIT. FOR A BETTER FINISH.